# TCL 210 Travel

**Bedienungsanleitung**  
**Operating Instructions**  
**Mode d'emploi**  
**Bedieningshandleiding**  
**Istruzioni d'uso**  
**Instrucciones de uso**

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- 1a Built-in flashlight
- 1b Switch for flashlight
- 2 Alarm mode switch  
(audio + shaker, shaker, audio, off)
- 3 MODE (alarm, timer, temperature)
- 4 SET to select setting choice
- 5 SNOOZE
- 6 UP ▲
- 7 DN ▼ /TEST
- 8 Eye for pillow clip
- 9 Audio alarm tone control (1-2-3 = low-medium-high)
- 10 Audio alarm volume control (1-2-3 = low-medium-high)
- 11 Battery Compartment (rear)

## Package Content

- Vibrating alarm clock
- 3 AA batteries
- Pillow clip (e. g. for fixing the clock to a pillow)
- User Guide

## Insert the batteries

Install three AA alkaline batteries (included).

Please observe battery polarity while installing.

A set of brand new alkaline batteries will last about eight to ten months of normal usage.

**Helpline UK 0844 800 6503**

(See chapter "Helpline" for costs and hours of operation)

## To set time

1. Press and hold **SET** for 3 seconds.  
 is flashing.
2. First, set the hour format (24 or 12 hours). Press **UP ▲** to adjust - press **SET** to confirm.
3. Press **UP ▲** or **DN ▼** to set the hour/minute as well as AM (morning) or PM (afternoon) and confirm each with **SET**.

## To set alarm time & snooze

1. Set the alarm mode switch to **ALL**.
2. Press **MODE** once to go to alarm time setting.  
 is flashing. Press and hold **SET** for 3 seconds to confirm.

3. Press **UP ▲** or **DN ▼** to set the hour/minute and confirm each with **SET**. After that **SNOZ** is flashing.
4. If you now press **SET** the snooze function is turned on.  
If you press **UP ▲** **SNOZ** disappears. If you now press **SET** the snooze function is turned off.
5. The clock will automatically return to the normal clock mode after about 15 seconds and the alarm is set.

## Note

If **snooze function is on**: when the alarm sounds off, the alarm will continue for 40 seconds and then stop. If **any button** is pushed any time

while the alarm is sounding off, the alarm will stop immediately but it will sound off again after eight minutes. You can repeat this cycle for up to seven times.

To stop the alarm completely at any time, press and hold the **SNOOZE** button for 3 seconds.

If **snooze function is off**: when the alarm sounds off, press any button to stop the alarm completely.

## **To set alarm on/off & alarm mode**

1. Slide the alarm mode switch to **OFF** to disable the alarm or to stop the on-going alarm action.

- Slide the switch to **AUDIO** () , **SHAKER** () or **ALL** (audio + shaker) to select your alarm mode and to set the alarm on.

## To test your alarm setting

- Press and hold **DN ▼ / TEST** for 3 seconds, the set alarm is triggered for a period of 10 seconds and then returns to the clock mode.

## To use clock as a Timer

- Press **MODE** twice to go to the timer mode (hours/minutes).  is flashing. Press and hold **SET** for 3 seconds to confirm.
- Press **UP ▲** or **DN ▼** to set the hour/minute and confirm each with **SET**.

**Helpline UK 0844 800 6503**

(See chapter "Helpline" for costs and hours of operation)

3. Press **DN** ▼ within 5 seconds to start the timer count-down.  , **H** and **M** are flashing and the time runs down. If **DN** ▼ is not pressed within 5 seconds the clock will automatically return to the normal clock mode.

**Note:** When the set time expires, the alarm will sound off (same alarm mode as the alarm clock setting) for 30 seconds (unless stopped by pressing any of the top keys). To stop the timer in between, press **DN** ▼ . The clock then will return to the normal clock mode.

## To show the room temperature

1. Press **MODE** three times to go to the thermometer mode.  is flashing and the room temperature appears.
2. If needed, press **UP ▲** to set reading in °C or °F
3. The clock will automatically return to the normal clock mode after about 5 seconds

**Helpline UK 0844 800 6503**

(See chapter "Helpline" for costs and hours of operation)

## **Specifications**

Functions: clock, alarm clock, timer, thermometer and flash-light

Clock modes: 12 or 24 hours

Alarm modes: audio, vibration, audio & vibration

Thermometer modes: °C or °F

Size: 118 x 71 x27 mm

Weight: (without batteries): 140 g

Power supply: 3 AA alkaline batteries (provided)

Battery life: 8-10 months of normal use

## **Safety notes**

Please make sure you read this user guide carefully and keep it for further use.

## Disposal

You are obliged to dispose of consumable goods properly in accordance with the applicable legal regulations. The symbol on this product indicates that electrical and electronic apparatus must be disposed of separately from domestic waste at suitable collection points provided by the public waste authorities. Packaging materials must be disposed of according to local regulations.

GB

## Helpline

If you need some help and you're in the UK, please call us on 0844 800 6503 (using an UK landline, at the time of going to print: calls cost 6p for connec-

**Helpline UK 0844 800 6503**

(See chapter "Helpline" for costs and hours of operation)

tion charge and then 5p a minute. Mobile costs may vary). Lines are open Monday to Friday, 9am to 5pm, excluding public holidays.

If you've a claim on your 2-year guarantee, please contact your supplier.

## **Maintenance**

- Please clean your equipment's surfaces with a soft, fluff-free cloth.
- Never use cleaning agents or solvents.

## **Guarantee**

AMPLICOMMS equipment is produced and tested according to the latest production methods. The implementation of carefully chosen materials and highly developed technologies ensure trouble free

functioning and a long service life.

The period of guarantee is 24 months from the date of purchase. The terms of the guarantee do not apply to the rechargeable batteries or power packs used in the products.

All deficiencies resulting from material of production faults which occur during the period of guarantee will be eliminated free of charge. Rights to claims under the terms of guarantee are annulled following intervention by the purchaser or third parties. Damage caused as a result of improper handling or operation, incorrect positioning or storing, improper connection or installation, Acts of God or other external influence are not covered by the terms of guarantee. In the case of complaints, we reserve the right to

**Helpline UK 0844 800 6503**

(See chapter "Helpline" for costs and hours of operation)

repair or replace defect parts or provide a replacement device. Replacement parts or devices become our property. Rights to compensation in the case of damage are excluded where there is no evidence of intent or gross negligence by the manufacturer.

If your equipment shows signs of defect during the period of guarantee, please return to the sales outlet in which you purchased the AMPLICOMMS equipment together with the purchase receipt. All rights to claims under the terms of guarantee in accordance with this agreement must be asserted exclusively with regard to your sales outlet.

Two years after the purchase of our products, claims under the terms of guarantee can no longer be asserted.

## Declaration of conformity

This device fulfils the requirements stipulated in the EU directive:

2004/108/EC relating to electromagnetic compatibility. Conformity with the above mentioned directive is confirmed by the CE symbol on the device. To view the complete Declaration of Conformity, please refer to the free download available on our website **[www.amplicomms.com](http://www.amplicomms.com)**.

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Audioline GmbH