Analogwecker mit schnurlosem Vibrationskissen
Analogue alarm clock with cordless vibration pad
Réveil analogique avec coussin vibrant inclus
Analoge wekker inclusief trilkussen

Bedienungsanleitung
Operating Instructions
Mode d'emploi
Gebruiksaanwijzing

D

GB

F

NL

TCL 190

Bedienungsanleitung	3
Operating Instructions	17
Mode d'emploi	31
Gebruiksaanwijzing	45

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Internet: www.amplicom.eu - E-mail: info@audioline.de

1 Safety Information

Please read this operating instruction manual thoroughly.

Intended use

This alarm clock is designed to provide an acoustic alarm or alert signal at a set time or on receipt of a telephone call. Any other use is considered unintended use. Unauthorised modification or reconstruction is not permitted. Under no circumstances open the device or complete any repair work yourself.

Installation location

Prevent excessive exposure to smoke, dust, vibration, chemicals, moisture, heat and direct sunlight.

Heart pacemaker

Attention: Always maintain a distance of at least 15 cm to implanted heart pacemakers to prevent any possible interference.

Power adapter plug

Attention: Only use the power adapter plug supplied because other power supplies could damage the equipment. Ensure access to the power adapter plug is not obstructed by furniture or such.

The power adapter plug supplied fulfils the eco-design directive of the European Council (Directive 2005/32/EC). This means that power consumption is considerably lower than power adapter plugs of an earlier design both in operating and idling modes.

Rechargeable batteries

Attention: Never throw batteries into a fire. Only use batteries of the same type.

Disposal

In order to dispose of your device, take it to a collection point provided by your local public waste authorities (e.g. recycling centre). According to laws on the disposal of electronic and electrical devices, owners are obliged to dispose of old **electronic and electrical devices** in a separate waste container. The adjacent symbol indicates that the device must not be disposed of in normal domestic waste!

Batteries represent a hazard to both health and the environment! Never

open, damage or swallow batteries, nor allow them to get into the environment.

They may contain toxic and ecologically hazardous heavy metals.

You are legally obliged to dispose of batteries at the point of sale or at appropriate collection points provided by your local public waste authorities. Disposal is free of charge. The adjacent symbols indicate that batteries must not be disposed of in normal domestic waste and but at authorised collection points.

Packaging materials must be disposed of according to local regulations.

Operating Elements


2 Operating Elements

Alarm clock (front view)

- 1 Snooze button
- 2 LED: Alarm ON/OFF / Battery charge low
- 3 Rotary knob: Switch alarm ON/OFF


Status indicators in clock face

- LED (Pos. 2) Alarm function is activated: LED is red
Battery charge low, alarm is activated: LED flashes green
Battery charge low, alarm is deactivated: LED flashes orange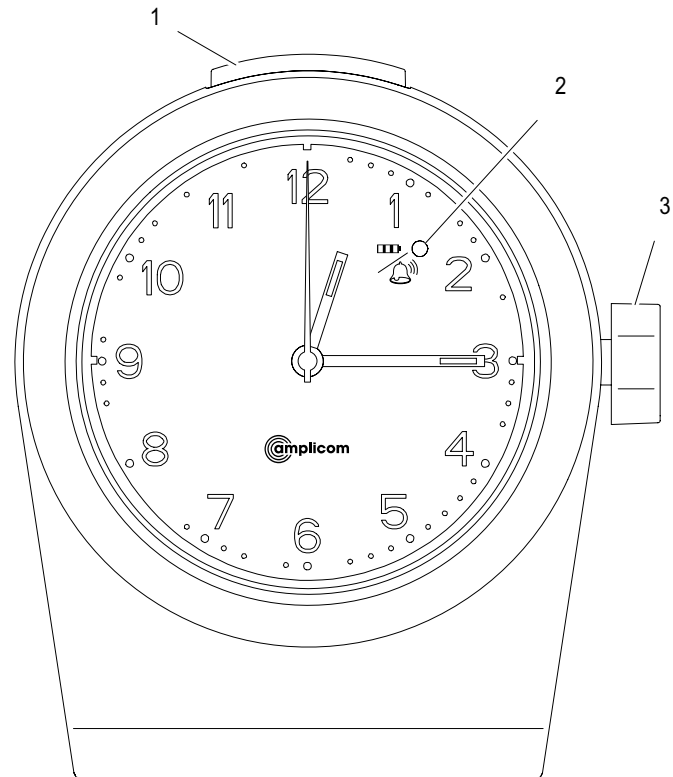


Alarm clock (rear view)

- 4 Rotary knob: Volume control
- 5 Rotary knob: Set one of the 5 acoustic signals
- 6 Rotary knob: Pitch
- 7 Sliding switch: "Vibration only", "Acoustic signal only", "Acoustic signal + Vibration"
- 8 Sliding switch: Flashing lamp ON/OFF
- 9 Sliding switch: Snooze delay, "0 min.", "5 min.", "15 min."
- 10 Sliding switch: Clock face illumination Bright/Medium/Off
- 11 Telephone line socket
- 12 Power adapter socket
- 13 Vibration pad socket
- 14 Rotary knob: Set alarm time
- 15 Rotary knob: Set time
- 16 Battery compartment (on underside)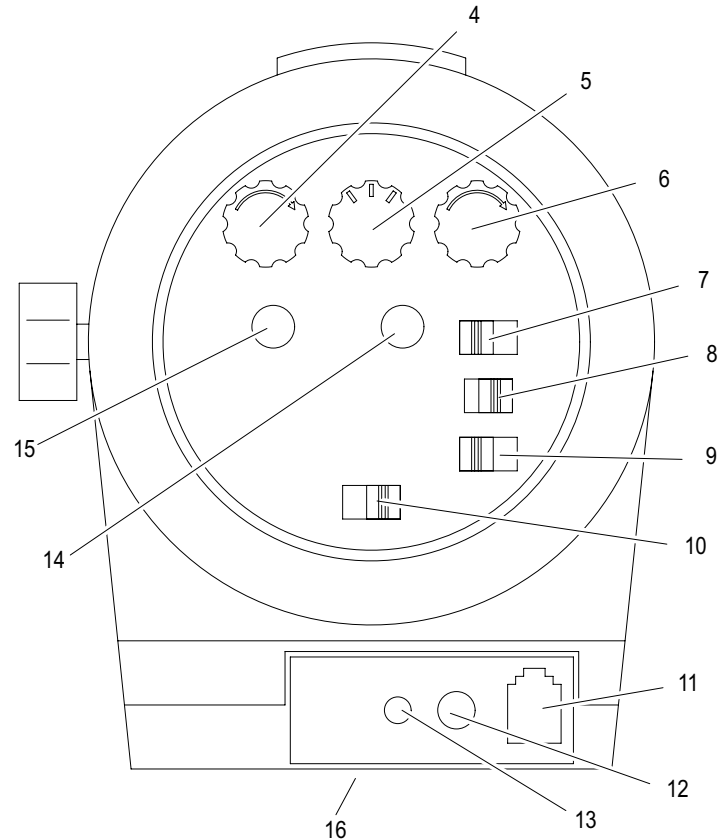

Operating Elements

Vibration pad

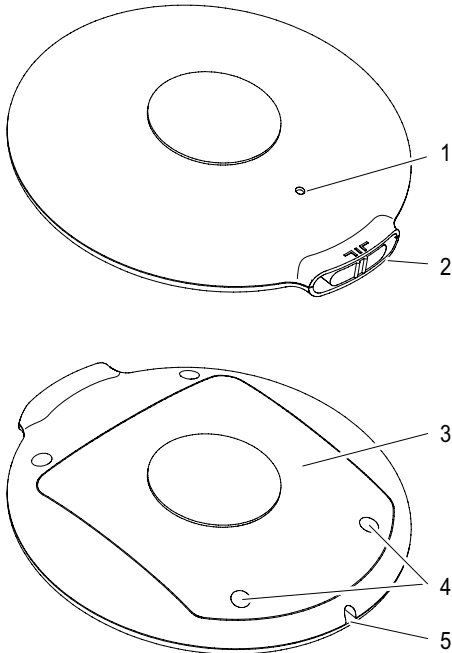- 1 LED: ON/OFF / Battery charge low
- 2 Sliding switch: Vibration pad ON /OFF
- 3 Battery compartment
- 4 Screws for battery compartment
- 5 Connection cable socket

Status indicators on vibration pad

- LED (Pos. 1) Switched on: LED flashes green every 2 sec.
 Charging process: LED is red
 Battery charge low: LED flashes red

3 Preparing the Equipment

Safety information

Attention: It is essential to read the Safety Information in Chapter 1 before starting up.

Checking the package contents


The package contains:

Content	TCL 190
Analogue alarm clock	1
Power adapter plug	1
Vibration pad	1
Connection cable for vibration pad	1
Batteries for analogue alarm clock	2
Batteries for vibration pad (integrated)	1
Telephone connection line	1
Operating instructions	1
Battery compartment cover	1

Connecting the alarm clock

Please proceed as follows in order to be able to use the alarm clock:

1. First of all, insert the batteries supplied in the clock's battery compartment.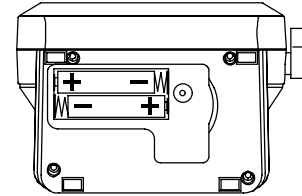

Pay attention to correct polarity when inserting the batteries.


2. Position the battery compartment cover on the underside of the alarm clock.
3. Screw the battery compartment cover to the alarm clock housing.

Preparing the Equipment

4. Connect the equipment as illustrated in the diagram.

Only use the telephone connection line supplied because the equipment may not function if another telephone connection line is used.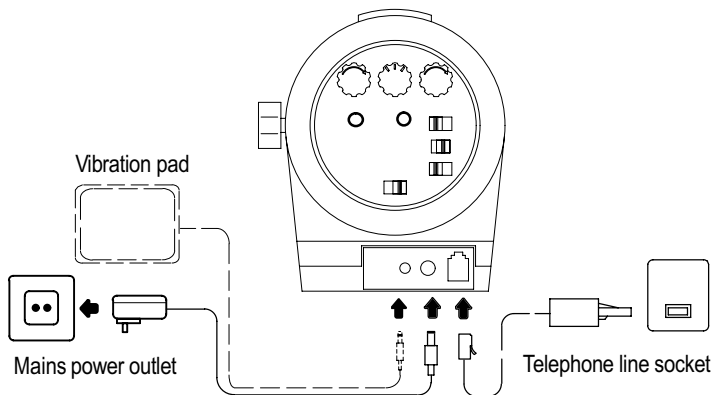

In order to connect the alarm clock to a telephone connection for use as a signal amplifier, you may need an appropriate adapter. You can obtain one at a specialist shop.

Only use the power adapter plug supplied labelled with output data 12 V_{DC}, 670 mA.

Basic settings

In order to be able to use the TCL 190 alarm clock, you must first carry out the following settings.

1. Set the time. Turn the rotary knob (15) until the clock shows the correct time.
2. Set the time at which the alarm should go off. Turn the rotary knob (14) until the red hand points to the time at which the alarm should be triggered.
3. Use the sliding switch (7) to select the type of alarm (mode). There are three settings available, from left to right "vibration only", "acoustic signal only" or "acoustic signal + vibration".
4. Turn the rotary knob (4) to set the required volume. Firstly, set the volume to "quiet" (turn anticlockwise) and increase the volume slowly.

Exposure to the alarm at full volume for a long period could cause ear damage.

5. Turn the rotary knob (5) to set the required acoustic signal.
6. Turn the rotary knob (3) to set the required pitch for the acoustic signal.

Use the rotary knobs for volume and pitch to select settings whereby the alarm can serve its purpose with the volume setting as low as possible. Depending on your hearing, it is possible that you can detect higher or lower pitch tones even at low volumes.

7. In order to activate the alarm to trigger at the set time, turn the rotary knob (3) clockwise to "On".

Operating the vibration pad

In order to use the vibration pad, it must initially be fully charged:

1. Connect the alarm clock to the mains power supply via the power adapter plug.
2. Remove the small rubber cap from the sliding switch on the vibration pad.
3. Plug one end of the connection cable in the socket in the vibration pad and in the other end in the socket for the vibration pad in the alarm clock.
4. The LED on the vibration pad lights up red.
5. When the vibration pad is fully charged, disconnect the connection cable from the alarm clock.
6. Use the sliding switch on the vibration pad to switch it on.
7. Set the alarm mode on the alarm clock to “vibration” or “acoustic alarm + vibration”.

On reaching the set time, the vibration pad vibrates intermittently until the alarm function is interrupted by pressing the Snooze button (1).

Using the telephone signal

You can also use the TCL 190 alarm clock to signal incoming telephone calls. Connect the TCL 190 alarm clock to the telephone connection socket as illustrated in the connection diagram.

Each incoming call is then diverted to the alarm clock and the acoustic signal is triggered by each call regardless of the alarm time set.

Settings and Functions

4 Settings and Functions

Setting the time


Use the rotary knob (15) to set the correct time.

Setting the alarm time

Use the rotary knob (14) to set the time at which the alarm should be triggered.

Switching the alarm signal on and off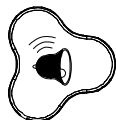


Activate or deactivate the alarm by turning the rotary knob (3) on or off, respectively. Anticlockwise "Off" or clockwise "On".

Setting the volume

The alarm and telephone ringer volumes start quietly and increase to the maximum volume.

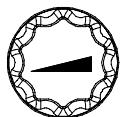Set the alarm volume by turning the volume rotary knob (4) to the required volume. First of all, set the volume to a quiet setting and increase it slowly.

Exposure to the alarm at full volume for a long period could cause ear damage.

Selecting the acoustic signal

Turn the rotary knob (5) to set the required acoustic signal.

Setting the pitch

Turn the rotary knob (3) to set the required pitch for the acoustic signal.

Setting the clock face illumination

Move the sliding switch (10) to activate the clock face illumination. The following settings are available, from left to right ("bright", "medium" or "off").


Selecting the type of alarm (mode)

Move the sliding switch (7) to set the type of alarm with which you want to be alerted. There are three settings available, from left to right ("vibration only", "acoustic signal only" or "acoustic signal + vibration").

Setting the flashing light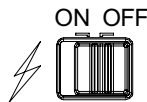

The sliding switch (8) is available with which to select whether the flashing lamp in the Snooze button should also be activated together with the alarm signal set. Two settings are available, from left to right "On" or "Off".

Setting the snooze delay

0m 5m 15m Use the sliding switch (9) to set the interval for the snooze delay. There are three settings available, from left to right "0 min", "5 min" or "15 min".

Activating the snooze delay


If the snooze button is pressed at anytime when the light is switched off, then the illumination will switch on for 30 seconds to allow the user to see the time. After 30 seconds the illumination will go out.

Press the Snooze button (1) to interrupt the alarm for a specific time before it automatically resumes. To activate the Snooze function, refer to "Setting the snooze delay".

Activating the vibration pad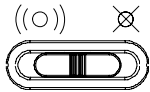

After having connected the vibration pad supplied, you can use the switch on the vibration pad to switch the vibration function on and off. There are two settings available, from left to right "On" or "Off".

In Case of Problems

5 In Case of Problems

Service-Hotline

Should problems arise with the device, please refer to the following information first. In the event of technical problems, please contact our service hotline under tel. 0844 800 6503 (Calls cost 6p for connection charge and then 5p a minute. Mobile costs may vary. Lines are open Monday to Friday, 9am to 5pm). In the case of claims under the terms of guarantee, contact your sales outlet. There is a 2 year period of guarantee.

Problems and solutions

Problems	Solutions
Equipment does not work	<ul style="list-style-type: none">- The battery charge is too low or no batteries are inserted.- The power adapter plug is not plugged in.
Vibration pad does not work	<ul style="list-style-type: none">- The battery charge is too low or the battery is defective.- Battery packs are available from Amplicom.- The vibration pad is not switched on.- It is outside the range of the alarm clock.
Alarm signal is not triggered	<ul style="list-style-type: none">- The alarm function is not switched on; turn the knob to "On".

6 Maintenance / Guarantee

Maintenance

- Clean the housing surfaces with a soft, fluff-free cloth.
- Never use cleaning agents or solvents.

Guarantee

AMPLICOM equipment is produced and tested according to the latest production methods. The implementation of carefully chosen materials and highly developed technologies ensure trouble-free functioning and a long service life. The terms of guarantee do not apply where the cause of equipment malfunction is the fault of the telephone network operator or any interposed private branch extension system. The terms of guarantee do not apply to the batteries or power packs used in the products. The period of guarantee is 24 months from the date of purchase.

All deficiencies related to material or manufacturing errors within the period of guarantee will be redressed free of charge. Rights to claims under the terms of guarantee are annulled following tampering by the purchaser or third parties. Damage caused as the result of improper handling or operation, normal wear and tear, incorrect positioning or storage, improper connection or installation or Acts of God and other external influences are excluded from the terms of guarantee. In the case of complaints, we reserve the right to repair defect parts, replace them or replace the entire device. Replaced parts or devices become our property. Rights to compensation in the case of damage are excluded where there is no evidence of intent or gross negligence by the manufacturer.

If your device does show signs of a defect within the period of guarantee, please contact the sales outlet where you purchased the AMPLICOM device, producing the purchase receipt as evidence. All claims under the terms of guarantee in accordance with this agreement can only be asserted at the sales outlet.

No claims under the terms of guarantee can be asserted after a period of two years from the date of purchase and hand-over of the product.

Declaration of Conformity

This device fulfils the requirements stipulated in the EU directive:

1999/5/EU directive on radio equipment and telecommunications terminal equipment and the mutual recognition of their conformity. Conformity with the above mentioned directive is confirmed by the CE mark on the device.

To view the complete Declaration of Conformity, please refer to the free download available on our web site www.amplicom.eu.

Technical Data

7 Technical Data

Analogue alarm clock	Value
Frequency:	433 MHz
Power supply:	Input: 210-240 VAC, 50/60 Hz Output: 12 VDC, 670 mA
Telephone plug:	BS 6312
Batteries:	2 x AAA 1.2 V, 500 mAh, NiMH
Initial charging time:	Up to 16 h
Permissible ambient temperature	0 °C to 40 °C
Permissible storage temperature:	-20 °C to 60 °C
Transmission power:	10 dBm
Dimensions L / W / H:	132 x 146 x 84.6 mm
Weight:	350 - 380 g

Vibration pad	Value
Range:	Approx. 100 m (clear space)
Battery:	3 x AAA 3.6 V, 800 mAh, NiMH
Initial charging time:	Up to 16 h
Permissible ambient temperature	0 °C to 40 °C
Permissible storage temperature:	-20 °C to 60 °C
Dimensions W / H:	113.3 x 36.6 mm
Weight:	180 - 200 g

Notes

Important Notes on the Guarantee

The period of guarantee for the AMPLICOM product begins on the date of purchase. Rights to claims under the terms of guarantee relate to deficiencies resulting from material or production faults. Rights to claims under the terms of guarantee are annulled following intervention by the purchaser or third parties. Damage caused as a result of improper handling or operation, normal wear and tear, incorrect positioning or storing, improper connection or installation, Acts of God or other external influence are not covered by the terms of guarantee.

CLAIMS UNDER THE TERMS OF GUARANTEE MAY ONLY BE ASSERTED TO THE RETAILER WHERE THE EQUIPMENT WAS PURCHASED, WHEREBY THE PURCHASE RECEIPT MUST ALSO BE SUBMITTED.

TO PREVENT UNNECESSARY LONG DELAYS, PLEASE DO NOT SEND EQUIPMENT UNDER COMPLAINT TO OUR OFFICE ADDRESS.

IN THE CASE OF QUESTIONS CONCERNING OPERATION OR THE TERMS OF GUARANTEE, PLEASE REFER TO THE OPERATING MANUAL OR CONTACT OUR HOTLINE: Helpline UK 0844 800 6503 (Calls cost 6p for connection charge and then 5p a minute. Mobile costs may vary. Lines are open Monday to Friday, 9am to 5pm)

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AUDIOLINE GmbH