CLA 1

GB User guide index p.2

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EXTRA EARPIECE

INDEX

DESCRIPTION	3
SETTING UP	
- Connecting the CLA 1	5
- Select your phone	5
- Select the working mode (Handset / headset)	5
- How to use the handset	6
- How to use the earpiece	6
- How to use the headset (option)	6
TROUBLE SHOOTING	7
GUARANTEE	8

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
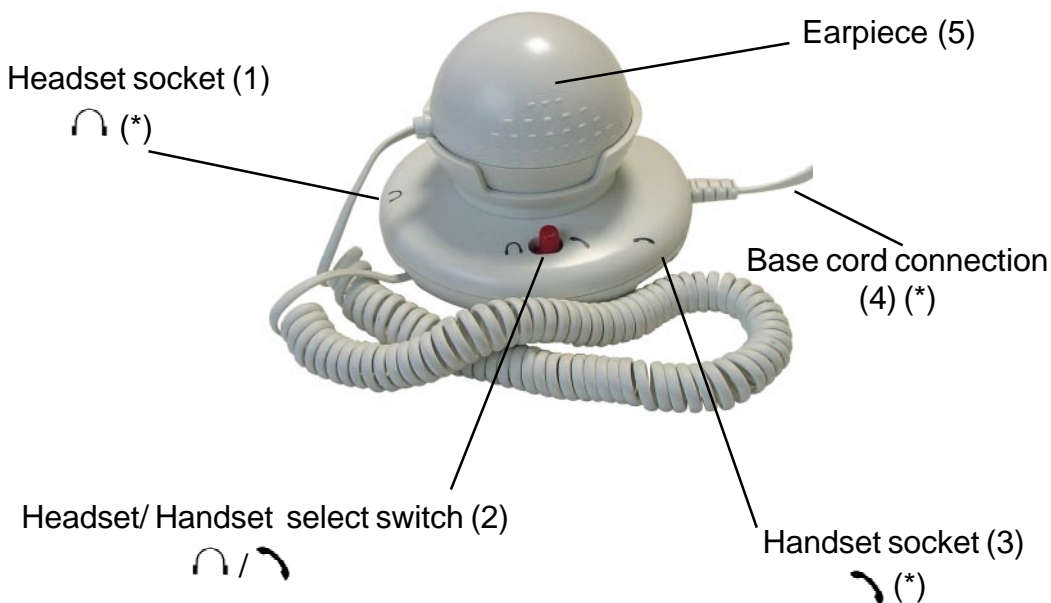
DESCRIPTION

Headset hook/Rack mounting

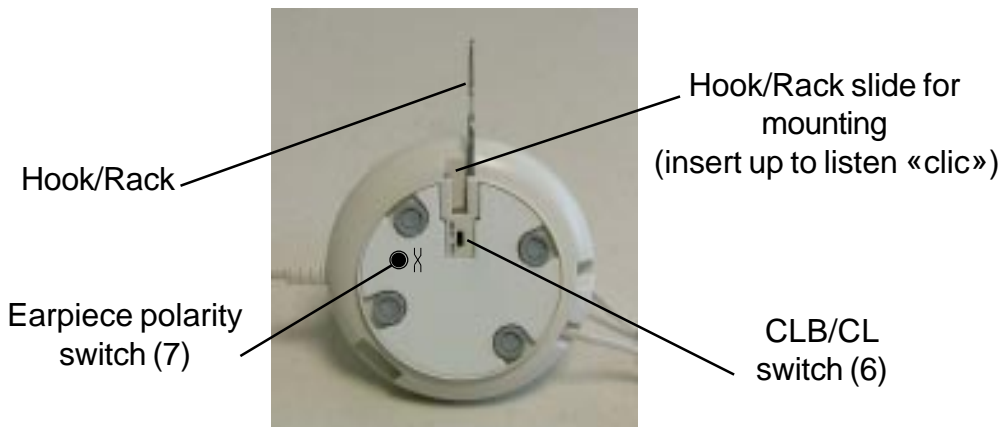(*) Classified TNV-3 according to EN60950 standard.

DESCRIPTION

View of the CLA 1 after Hook/Rack mounted:

View of the CLA 1 with a headset on Hook/Rack:

Note: Headset is an option and is not supplied with the CLA 1.
Geemarc headset reference : **HEADSCL_BLK**

SETTINGS

SETTING

Connecting the CLA 1

To connect the **CLA 1** to your phone please follow the instructions carefully. See pictures on page 3 to locate the plugs and socket.

- 1 - Disconnect your handset from your phone base.
- 2 - Connect the **CLA 1** short curly cord (4) to the handset phone socket.
- 3 - Connect the handset curly cord to the **CLA 1** handset socket  (3) .
- 4 - Connect the headset (option) to the headset socket  (1) .

Select your phone

To have the best receiving level set the switch **CL-B/CL** (6) located under the **CLA 1** to the desired position, the **CL-B** position must be used for phones with handset boost feature (like **CL 200, CL 400, CL 500**) and **CL** position for normal phone (like **CL 50, CL 100**).

Select the function mode (Handset / headset)

You can select the function mode by using the switch (2) located on the **CLA 1** top to select the handset by setting the switch to the  position and the headset by setting the switch to the  .

The **CLA 1** is now ready to use.

Note: The **CLA 1** functionality are only guaranteed with Geemarc telephones **CL 50 to CL 500** range.

SETTINGS

How to use the handset

Make sure that the switch (2) is set to  . When you receive a call lift the handset from the cradle and speak, at the end of the call replace the handset in the phone cradle.

How to use the earpiece

When you receive a call, lift the handset from the cradle and speak. A second person can lift the earpiece and use it as a handset to listen to the conversation. Or you can put the earpiece to your other ear to improve your reception. At the end of the call simply replace the handset and the earpiece in their respective cradles.

Note: In this case, earpiece and handset can be used at the same time.

How to use the headset (headset is not supplied)

When you receive a call, lift your handset and place it near the phone base, make sure that the switch (2) is set to  . Then speak and listen your correspondent with your headset (Geemarc headset ref: **HEADSCL_BLK**) . At the end of the call replace the handset in the phone cradle.

Note: In this case, earpiece and headset can be used at the same time.

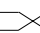
TROUBLE SHOOTING

IMPORTANT NOTICE:

The **CLA 1** will work with all **CLEAR SOUND 50** to **500** series. It may happen that **CLA 1** does not work with telephones from others suppliers, we cannot guarantee the functionality of the **CLA 1** in this case.

Note: if you get a loud noise oscillation or crackling when connecting the headset simply press the switch  (7) located under the **CLA 1** to avoid the problem (earpiece polarity).

TROUBLE SHOOTING

- No dial tone: Make sure everythings is connected in the correct plugs.
- The receiving level is too low or too high: Check the **CL-B/CL** switch located under your **CLA 1**.
- Crackling into the headset or earpiece: check the  switch (7) located under the **CLA 1**.
- Nothing happens after following the instructions above: check your telephone and the network connection.

For product support and help visit our website at www.geemarc.com

telephone 01707 384438

or fax 01707 372529

GUARANTEE

GUARANTEE

From the moment your Geemarc product is purchased, Geemarc guarantee it for the period of one year. During this time, all repairs or replacements (at our option) are free of charge. Should you experience a problem then contact our helpline or visit our website at www.geemarc.com. The guarantee does not cover accidents, negligence or breakages to any parts. The product must not be tampered with or taken apart by anyone who is not an authorised Geemarc representative. The Geemarc guarantee in no way limits your legal rights.

IMPORTANT: YOUR RECEIPT IS PART OF YOUR GUARANTEE AND MUST BE RETAINED AND PRODUCED IN THE EVENT OF A WARRANTY CLAIM

Please note: The guarantee applies to the United Kingdom only.

DECLARATION: Geemarc Telecom SA Hereby declares that this product is in compliance with the essential requirements and other relevant provisions of the Radio and Telecommunications Terminal Equipment Directive 1999/5/EEC and in particular article 3 section 1a, 1b and section 3.

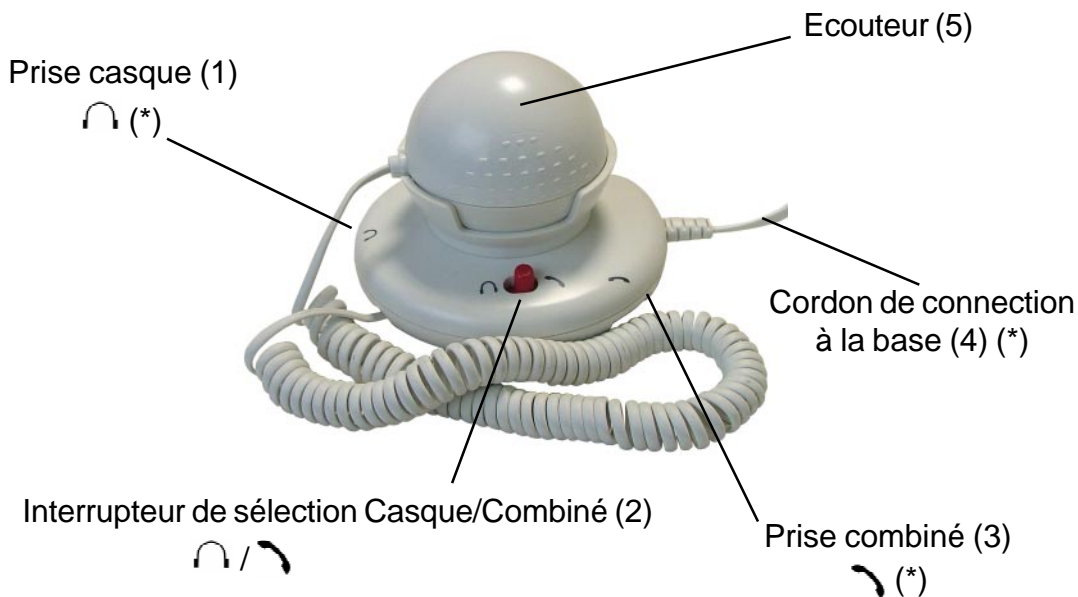
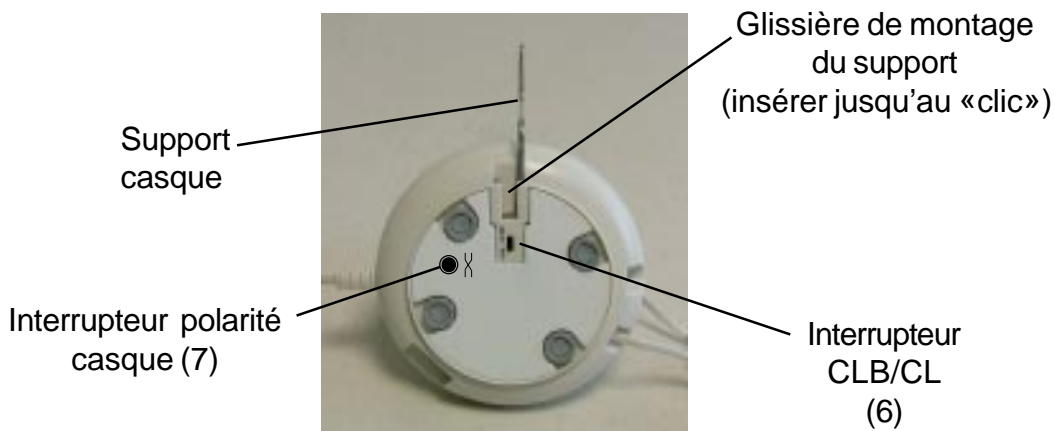
For product support and help visit our website at www.geemarc.com

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