



PowerTel 701

**Bedienungs-
anleitung**

User manual

**Mode
d'emploi**

**Gebruikers-
handleiding**




DE

GB

FR

NL

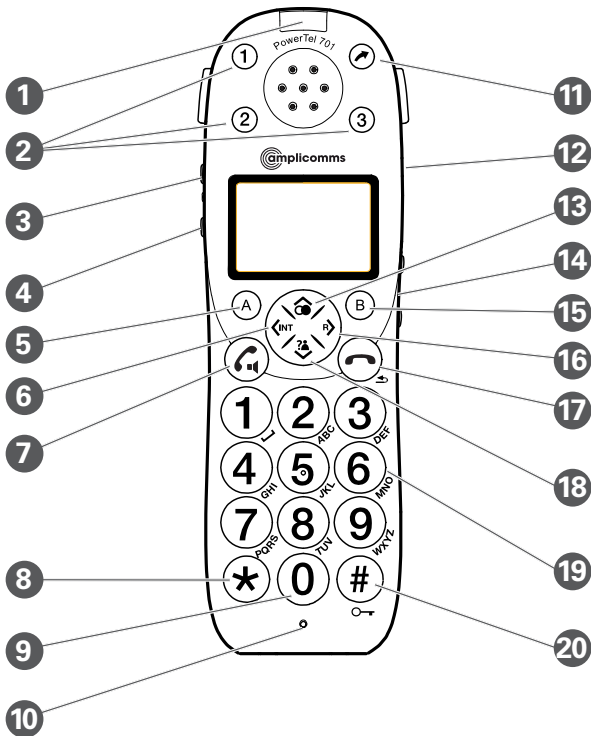

- 1 Incoming call light
- 2 Memory 1, 2, 3
- 3 Boost button with LED
- 4 Profile setting
- 5 Left softkey (opens the menu in standby state)
- 6 Internal key
- 7 External call/ Handsfree
- 8 ★/Ringer on/off
- 9 Pause (long press 0)
- 10 Microphone
- 11 Call transfer
- 12 Headset socket (Headset not included)
- 13 Up/Redial
- 14 Volume (+ / -)
- 15 Phonebook/ Save/OK
- 16 Recall
- 17 End call
- 18 Down/Calls list/ Equalizer
- 19 Keypad
- 20 #/Keylock


Connect your charger base

Plug the other power adapter that's already attached to your charger base into your mains power wall socket and switch the power on. You should ensure access to the power adapter plug is not obstructed by furniture or such.


Registering the handset

Important: make sure any handset you're registering is fully charged before you try to register it.

At the handset:

1. Press and hold  for approx 10 seconds until the display shows **Press & Hold Base Page Key**.

At the base:

2. Press and hold  for at least 10 seconds until the indicator light on the base flickers.

Detailed information for using the handset and instructions for safety and disposal you find in the **user guide of your PowerTel 700/702**.



Menu Structure with a base PowerTel 700/702

Ans Machine	—	Play New Msgs	
		Play All Msgs	
Phonebook			
Profile			
Handset	—	Ring Tone	External / Internal
		Ringer Volume	
		Equalizer	
		Boost On	
		Handset Name	
		Keypad Beep	
		Contrast	
		Auto Talk	
		Backlight Time	
		Confirm Beep	
		Select Base	
		Language	
		Registration	
Base Settings	—	Ring Tone	
		Ringer Volume	
		Dial Mode	Tone / Pulse
		Recall	Short / Long
		System PIN	
Time	—	Set Alarm	
		Date and Time	
		Set Time Format	
Tools	—	Appointment	
		Stop Watch	





Distributed by: Audioline GmbH, D-41460 Neuss
August 11 – ISSUE 1.

