


AMPLIDECT™ COMBI 295


QUICK USER GUIDE

En


Base

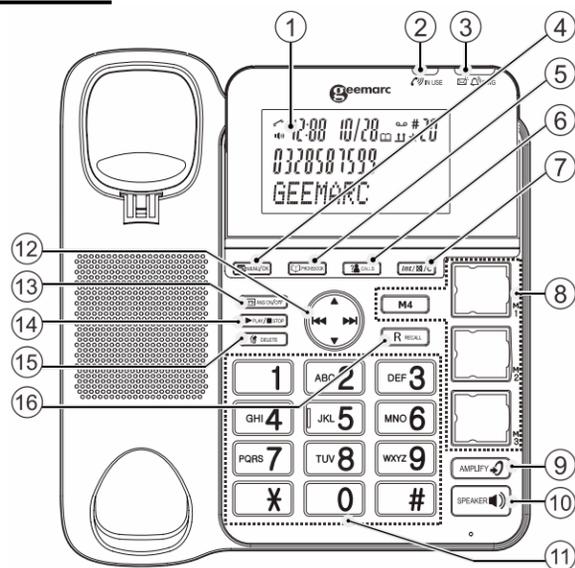


Handset

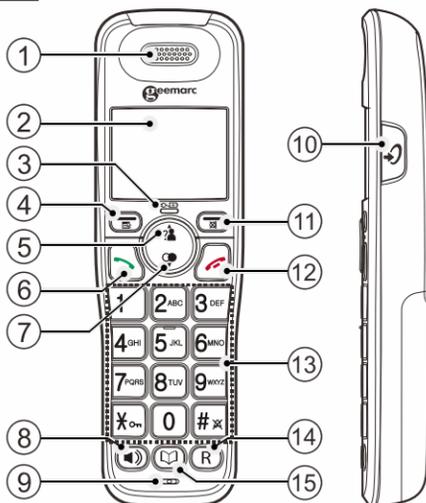

Handset

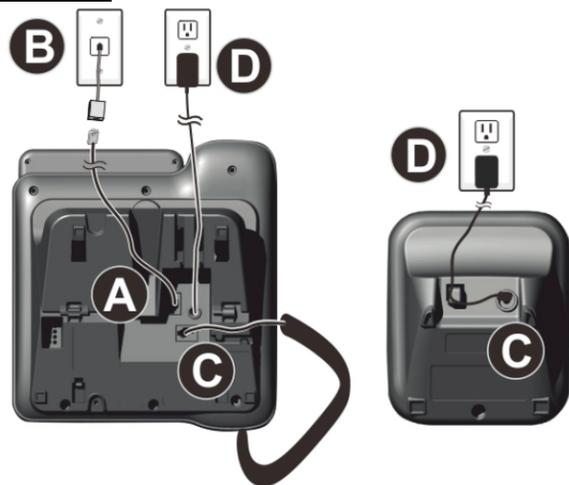

Wall Mounting

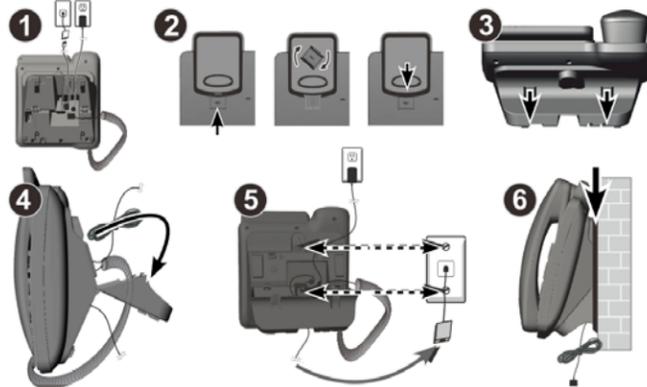

Please check Geemarc.com website to get the detailed User instruction manual

Congratulations on purchasing your Geemarc AMPLIDECT COMBI 295 telephone. This telephone offers all the advantages of cordless technology (compliant with Digitally Enhanced Cordless Telecommunications-DECT standard) together with an easy to see big button keypad, ideal for those who suffer with poor sight. This telephone is also hearing aid compatible.

It is important that you read the instructions below in order to use your Geemarc telephone to its full potential.

PACKAGE CONTENTS

1 Telephone base, 1 Corded handset, 1 cordless Handset, 2 power adapter, 1 Charger, 1 Telephone line cord, 1 NiMh battery pack, 1 Quick user guide.

INSTALLATION

Connecting the telephone

Connect the telephone cable to the telephone line socket (A) and to the telephone wall socket (B)

Connect the handset curly cord to the base.

Connect the power adapter jack to base and charger (C) then connect the power adapter plug in mains power (D).

Note: Use only the power adapters supplied with the phone.

Connecting the battery

Plug the battery connector securely into the socket inside the handset battery compartment.

Insert the supplied battery as indicated.

Align the cover flat against the battery compartment, and then slide it upwards until it clicks into place.

Charging the batteries

Charge the batteries for at least 12 hours when charging for the first time.

OVERVIEW

Base

1- LCD DISPLAY

2- IN USE (📞)

On during a call.

3- RING light (📞🔔)

Flashes when there is an incoming call.

Steadily on when the answering machine is turned on. Flashes when there are new memos or messages in the answering machine.

4- MENU/OK (📁)

In idle mode: Press to access the main menu.

In sub-menu mode: Press to confirm the selection.

In Redial list / Call list: Press to store the number into Phonebook.

5- PHONEBOOK (📖)

In Idle: Press to access the phonebook.

6- CALLS (CALL LIST) (👤)

In Idle mode: Press to access the call list.

7- INT/MUTE/CLEAR (Int/🔇/C)

In main menu mode: Press to go back to idle screen.

In sub-menu mode: Press to go back to previous level.

In sub-menu mode: Press and hold to go back to idle screen.

In editing / predialing mode: Press to clear a character / digit.

In editing / predialing mode: Press and hold to delete all the character / digits

During a call: Press to mute / unmute the microphone.

In ringing: Press to silence the base ringer.

In idle mode: Press to intercom another system handset.

8- MEMORY KEYS (3 PHOTO SPEED DIAL KEYS; 1 MEMORY KEY - M4)

In predial mode: Press and hold to store the number entered from idle mode.

In idle mode: Press to dial out to the number stored in the memory key.

9- VOLUME BOOST (🔊)

During a call: Press to activate/deactivate the VOLUME BOOST function.

10- SPEAKER (🔊)

In Idle mode or Phonebook/ Call list/ Redial list view mode: Press to make a call with speakerphone.

11- ALPHANUMERIC KEYPAD, STAR (*), HASH (#)

In predial or editing mode: Press to insert a digit

0 key in predial or editing mode: Press and hold to insert a pause.

12- NAVIGATION KEYS

UP (▲)

In menu mode: Press to scroll up the menu items.

In Phonebook list / Redial list / Call list: Press to scroll up the list.

In editing mode: Press to move the cursor one character to the left.

During a call or TAM message playback: Press to increase the volume.

REPEAT/OGM (⏮)

In Idle mode: Press and hold to record OGM.

Press to playback OGM.

In TAM message playback mode: Press to go back to previous message.

MEMO/SKIP FORWARD (⏭)

In Idle mode: Press and hold to record a memo.

In TAM message playback mode: Press to skip to next message.

DOWN/REDIAL LIST (▼)

In idle mode: Press to access the redial list.

In menu mode: Press to scroll down the menu items.

In Phonebook list / Redial list / Call list view mode: Press to scroll down the list.

In editing mode: Press to move the cursor one character to the right.

During a call or TAM message playback: Press to decrease the volume.

13-ANS ON/OFF (📞)

In Idle: Press to switch the answering machine ON or OFF.

14. PLAY/STOP (▶/■)

In idle mode: Press to playback messages.

During TAM message playback: Press to stop playing messages.

15. DELETE (🗑)

In TAM message playback mode: Press to delete the current TAM message playback.

In idle mode: Press and hold to delete all old messages.

16. RECALL (📞)

During a call: Press to activate Recall for use with Telephone Network Services such as Call Waiting during a call.

During message playback: Press to call back the caller if the caller's number is available

Handset

1- EARPIECE

2. LCD DISPLAY

3. CHARGE LIGHT (🔌)

Steadily on when the handset is charging in the charger.

4. MENU/OK (📁)

In idle mode: Press to access the main menu.

In sub-menu mode: Press to confirm the selection.

During a call: Press to access Intercom / Phonebook / Redial list / Call list.

5. UP/CALL LIST (👤)

In idle mode: Press to access the call list.

In menu mode: Press to scroll up the menu items.

In phonebook list/redial list/call list: Press to scroll up the list.

In editing mode: Press to move the cursor one character to the left.

During a call: Press to increase the listening volume.

During ringing: Press to increase the ringer volume.

6. TALK ON (📞)

In idle/pre-dial mode: Press to make a call in earpiece mode.

In phonebook/redial list/call list: Press to make a call to the selected entry in the list in earpiece mode.

During ringing: Press to answer a call in earpiece mode.

7. DOWN/REDIAL LIST (📞)

In idle mode: Press to access the redial list.

In menu mode: Press to scroll down the menu items.

In phonebook list/redial list/call list: Press to scroll down the list.

In editing mode: Press to move the cursor one character to the right.

During a call: Press to decrease the listening volume.

During ringing: Press to decrease the ringer volume.

8. SPEAKERPHONE (🔊)

In idle/pre-dial mode: Press to make a call in speakerphone mode. Press again to switch to earpiece mode.

In phonebook list/redial list/call list: Press to make a call to the selected entry in the list in speakerphone mode. Press again to switch to earpiece mode.

During ringing: Press to answer a call in speakerphone mode. Press again to switch to earpiece mode.

9. MICROPHONE

10. VOLUME BOOST (🔊)

During a call: Press to activate/deactivate the VOLUME BOOST function.

11. CANCEL/MUTE (🔇)

In main menu mode: Press to go back to idle screen.

In sub-menu mode: Press to go back to previous menu.

In sub-menu mode: Press and hold to go back to idle screen.

In editing/pre-dial mode: Press to delete a character/digit.

In editing/pre-dial mode: Press and hold to delete all the characters/digits.

During a call: Press to mute/resume the microphone.

During ringing: Press to silence the handset ringer.

In idle mode: Press to intercom another handset.

12. TALK OFF (📞)

During a call: Press to end a call and go back to idle screen In menu/editing mode: Press to go back to previous menu.

In menu/editing mode: Press and hold to go back to idle screen.

In idle mode: Press and hold to power off the handset.

Press and hold to power on when it is off.

13. ALPHANUMERIC KEYPAD, STAR (*), HASH (#)
 In predial or editing mode: Press to insert a digit / character /*/#.
 * key in idle mode: Press and hold to enable or disable the keypad lock.
 # key in idle mode: Press and hold to turn on/off the ringer.
 0 key in idle/pre-dial mode/number editing mode: Press and hold to insert a pause.

14. R
 During a call: Press to answer a new call if you subscribed to Call Waiting service from your service provider.

15. PHONEBOOK ()
 In idle mode: Press to access the phonebook.

The phone settings are always set in the same manner:
 Press **Menu/OK** () button to open the main menu. Then use the **▼▲** buttons to select an entry and confirm it by pressing the **Menu/OK** () button.

The factory preset **PIN** code is **0000**

Example:
Set the language on the base
 Press **MENU / OK** (), then select **▼▲** language, press **MENU / OK** (), then select **▼▲** desired language and then press **MENU / OK** () to confirm.

Set the language on the handset
 Press **MENU / OK** (), then select **▼▲** HS settings, press **MENU / OK** (), then select **▼▲** language, press **MENU / OK** (), then select **▼▲** desired language and then press **MENU / OK** () to confirm.

MENU SETTING

Base
PHONEBOOK
 ►VIEW ►ADD ►EDIT ►DELETE ►DELETE ALL ►PB STATUS
CALL LOG
 ►MISSED ►RECEIVED ►DIALLED
DATE & TIME
 ►TIME FORMAT ►SET CLOCK
LANGUAGE
 ►ENGLISH ►DEUTSCH ►FRANÇAIS ►ITALIANO
 ►NEDERLANDS...
SETTINGS
 ►RING SETUP ►DELETE HS ►DIAL MODE ►FLASH TIME
 ►KEY TONE ►CHANGE PIN ►BS DEFAULT

PAGE ALL HS
REGISTRATION

ANS. MACHINE
 ►MSG PLAYBACK ►DELETE ALL ►MEMO ►TAM ON/OFF
 ►TAM SETTINGS (ANSWER MODE, OGM SETTINGS, ANSWER DELAY, RECORD TIME, TAM LANGUAGE, BS SCREENING, REMOTE ACC, COMPRESSION, CHANGE PIN, TAM DEFAULT)

Handset

PHONEBOOK
 ►VIEW LIST
 under an entry: ►ADD ►VIEW ►EDIT ►DELETE ►DELETE ALL ►PB STATUS ►FASTDIAL
CALL LIST
 ►VIEW LIST
 under an entry: ►ADD TO PB ►DELETE ►DELETE ALL ►DETAILS ►CATEGORY

BS SETTINGS
 ►BS RINGER ►RING VOLUME ►DELETEHS
 ►DIAL MODE ►FLASH TIME►CHANGE PIN
HS SETTINGS
 ►RING SETUP►TONE SETUP►SELECT BASE
 ►LANGUAGE ►RENAME HS ►HS DISPLAY ►AUTO ANSWER

DIRECT MEM
 ►M1-M2-M3-M4
DATE & TIME
 ►DATE FORMAT ►TIME FORMAT ►SET TIME ►SET DATE

ALARM
 ►ON ►OFF

REGISTRATION
 ►BASE 1, 2, 3 or 4

DEFAULT
 ►PIN? ----

ANS. MACHINE
 ►MSG PLAYBACK ►DELETE ALL ►MEMO ►TAM ON/OFF
 ►TAM SETTINGS (ANSWER MODE, TAM LANGUAGE, OGM SETTINGS, ANSWER DELAY, RECORD TIME, COMPRESSION, BS SCREENING,REMOTE ACC, CHANGE PIN)

REGISTRATION & DE-REGISTRATION

Note: Handset and base station are pre-registered.
 Up to five handsets can be registered to a single base station.
 In the case of re-registration, the PIN code is **0000**.

Registration a handset
On Handset :Press **MENU/OK** () select **▼▲** REGISTRATION, Press **MENU/OK** () select **▼▲**BASE(1-4), Press **MENU/OK** () enter PIN.

At the same time on **Base:** Press **MENU/OK** (), select **▼▲** REGISTRATION, press **MENU/OK** () to see PLEASE WAIT (If the registration is successful, there will be a confirmation beep).

De-Registration
On Base: Press **MENU/OK** (), select **▼▲**SETTINGS, press **MENU/OK** () select **▼▲**DELETE HS, press **MENU/OK** () enter PIN, press **MENU/OK** () select **▼▲**one of the handsets that you want to de-register, press **MENU/ OK** () to confirm.

INTERCOM

Internal Call
Base: Press **INT/☒/CLEAR (Int/☒/C)**, select **▼▲**HS 1 - 5, press **MENU /OK** () to connect.
Handset: Press **C / MUTE** (), Enter the desired handset number on keypad.
 Base is "7"

Call All Handsets
Base: Press **INT/☒/CLEAR (Int/☒/C)**, select **▼▲**ALL HANDSETS, press **MENU /OK** () to connect.
Handset: Press **C / MUTE** (), Enter "9" on keypad.

Transfer an External Call
Base: Press **MENU /OK** (), select **▼▲**handset number, press **☒** or **☒** on handset. After connection quit the call on base.
Handset: Press **MENU / OK** (), select **▼▲**INTERCOM, Press **MENU / OK** (),

Enter the desired handset number on keypad. After connection quit the call on handset.

Conference Call

Handset: Press **MENU / OK** (), select **▼▲**INTERCOM, Press **MENU / OK** (),
 Enter the desired handset number on keypad.
 After connection press and hold ***** until CONFERENCE is shown.

Your telephone base includes a telephone answering machine that records unanswered calls when it is on. The answering machine can store up to 59 messages within the maximum recording time of approximately 15 minutes. As well as recording incoming messages, you can record memos for other users of your phone.

Press **☒ANS ON/OFF** to switch the answering machine ON or OFF

REMOTE OPERATIONS

If you are away from your home, you can call your answering machine to switch on or off the answering machine and listen to your messages remotely using another tone-dialing phone.

- Place a call from a tone-dialing phone.
- When the answering machine answers the call and starts playing OGM, press *****.
- Enter the 4-digit Remote Access PIN (default PIN is **0000**).
- Press the following keys to carry out your desired function.

A	Functions	
	While message is not playing	While message is playing
2		Delete the current message playback
4		Repeat playing the current message from the beginning. Press twice to skip backward to the previous message.
5	Play the message	
6		Skip forward to play the next message
7	Turn on the answering machine	
8		Stop the current message playback
9	Turn off the answering machine	
*	Enter the 4-digit remote access PIN	

NOTE:
 When your answering machine memory is full and the answer mode is in ANS & RECORD, the answer mode will change to ANSWER ONLY automatically.
 It will change to ANS & RECORD mode automatically after some messages have been deleted.
 You cannot delete new messages before they were listened

MAINTENANCE

Clean the surface of the housing with a soft, fluff-free cloth. Do not use any cleaning agents or solvents.

GUARANTEE

From the moment your Geemarc product is purchased, Geemarc guarantee it for the period of two years. During this time, all repairs or replacements (at our discretion) are free of charge. Should you experience a problem then contact our help line or visit our website at **www.geemarc.com**.

The guarantee does not cover accidents, negligence or breakages to any parts. The product must not be tampered with or taken apart by anyone who is not an authorized Geemarc

representative.
 The Geemarc guarantee in no way limits your legal rights.
IMPORTANT: YOUR RECEIPT IS PART OF YOUR GUARANTEE AND MUST BE RETAINED AND PRODUCED IN THE EVENT OF A WARRANTY CLAIM.
 Please note: The guarantee applies to the United Kingdom only.

DECLARATIONS:

CE Directives
 Geemarc Telecom SA hereby declares that this product is in compliance with the essential requirements and other relevant provisions of the Radio and Telecommunications Terminal Equipment Directive 2014/53/UE and in particular article 3 section 1a,1b and section 2. The telephone does not operate if the line current is lower than 18mA.
 The CE declaration of conformity may be consulted at **www.geemarc.com**

Electrical connection: The apparatus is designed to operate from a 230V 50Hz supply only. (Classified as hazardous voltage according to EN60950 standard). The apparatus does not incorporate an integral power on/off switch. To switch off the apparatus, disconnect the power adaptor.

When installing the apparatus, ensure that the mains power socket is readily accessible.

Telephone connection:
 Voltages present on the telecommunication network are classified TNV-3 (Telecommunication Network Voltage) according to the EN60950 standard.

Recycling Directives
 The WEEE (Waste Electrical and Electronic Equipment) has been put in place for the products at the end of their useful life are recycled in the best way.

When this product is finished with, please do not put it in your domestic waste bin. Please use one of the following disposal options:

- Remove the batteries and deposit them in an appropriate WEEE skip. Deposit the product in an appropriate WEEE skip.
- Or, hand the old product to the retailer. If you purchase a new one, they should accept it.

Thus if you respect these instructions you ensure human health and environmental protection.



For product support and help visit our website at **www.geemarc.com**
 telephone **01707 387602**
 or fax **01707 832529**

