



## QUICK USER GUIDE

En


### Base

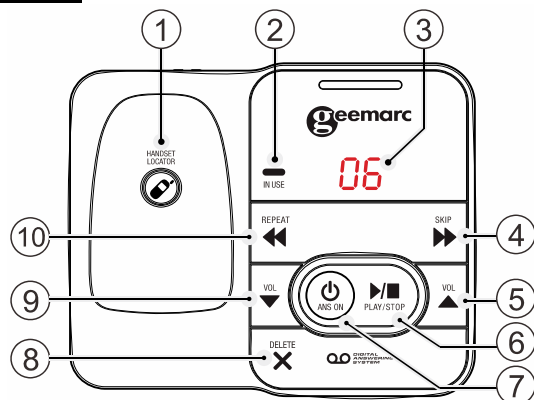


### Handset

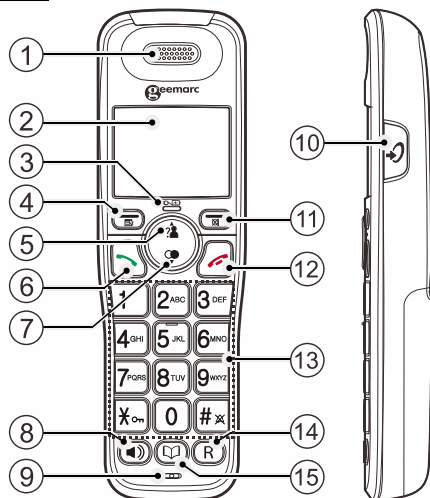

### Connection base

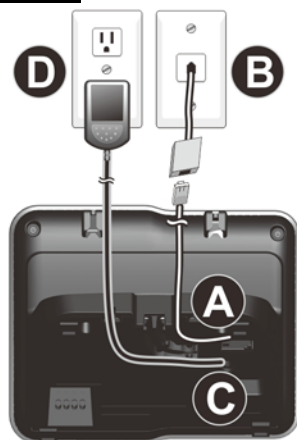

### Connection additional handset (Option)

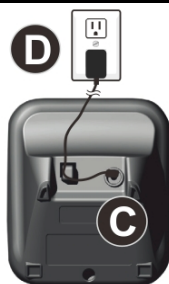

Please check [Geemarc.com](http://Geemarc.com) website to get the detailed User instruction manual

## PACKAGE CONTENTS

1 Telephone base, 1 cordless Handset, 1 power adapter, 1 Telephone line cord, 1 NiMh battery pack, 1 Quick user guide.

## INSTALLATION

### Connecting the telephone

Connect the telephone cable to the telephone line socket (A) and to the telephone wall socket (B).

Connect the power adapter jack to base (and DC charger) (C) then connect the power adapter plug in mains power (D).

Note: Use only the power adapters supplied with the phone.

### Connecting the battery

Plug the battery connector securely into the socket inside the handset battery compartment.

Insert the supplied battery as indicated.

Align the cover flat against the battery compartment, and then slide it upwards until it clicks into place.



### Charging the batteries

Charge the batteries for at least 12 hours when charging for the first time.

## OVERVIEW

### Base

#### 1- /HANDSET LOCATOR

While the phone is idle, press to page all handsets.

#### 2- IN USE

ON when handsets are in use, or when the answering system is answering an incoming call.

ON when a handset is being registered.

Flashes when there is an incoming call or when handsets are being deregistered.

Flashes when another telephone is in use on the same line.

#### 3- MESSAGE COUNTER

Flashes when there is an incoming call.

#### 4- /SKIP

Press to skip a message.

#### 5- VOL (▲)

During message playback, press to increase the listening volume. While in idle mode, press to increase the base ringer volume.

#### 6- /PLAY/STOP

Press to start or stop message playback.

#### 7- /ANS ON

Press to turn the built-in answering system on or off.

#### 8- /DELETE

Press to delete the message currently playing.

Press twice to delete all old messages when the phone is not in use.

#### 9- VOL (▼)

During message playback, press to decrease the listening volume. While in idle mode, press to decrease the base ringer volume.

#### 10- /REPEAT

Press to repeat a message.

Press twice to play the previous message.

### Handset

#### 1- EARPIECE

#### 2. LCD DISPLAY

#### 3. CHARGE LIGHT ( )

Steadily on when the handset is charging in the charger.

#### 4. MENU/OK ( )

In idle mode: Press to access the main menu.

In sub-menu mode: Press to confirm the selection.

During a call: Press to access Intercom / Phonebook / Redial list / Call list.

#### 5. UP/CALL LIST ( ? )

In idle mode: Press to access the call list.

In menu mode: Press to scroll up the menu items.

In phonebook list/redial list/call list: Press to scroll up the list.

In editing mode: Press to move the cursor one character to the left.

During a call: Press to increase the listening volume.

During ringing: Press to increase the ringer volume.

#### 6. TALK ON ( )

In idle/pre-dial mode: Press to make a call in earpiece mode.

In phonebook/redial list/call list: Press to make a call to the selected entry in the list in earpiece mode.

During ringing: Press to answer a call in earpiece mode.

#### 7. DOWN/REDIAL LIST ( )

In idle mode: Press to access the redial list.

In menu mode: Press to scroll down the menu items.

In phonebook list/redial list/call list: Press to scroll down the list.

In editing mode: Press to move the cursor one character to the right.

During a call: Press to decrease the listening volume.

During ringing: Press to decrease the ringer volume.

#### 8. SPEAKERPHONE ( )

In idle/pre-dial mode: Press to make a call in speakerphone mode. Press again to switch to earpiece mode.

In phonebook list/redial list/call list: Press to make a call to the selected entry in the list in speakerphone mode. Press again to switch to earpiece mode.

During ringing: Press to answer a call in speakerphone mode. Press again to switch to earpiece mode.

#### 9. MICROPHONE

#### 10. VOLUME BOOST ( )

During a call: Press to activate/deactivate the VOLUME BOOST function.

#### 11. CANCEL/MUTE ( )

In main menu mode: Press to go back to idle screen.

In sub-menu mode: Press to go back to previous menu.

In sub-menu mode: Press and hold to go back to idle screen.

In editing/pre-dial mode: Press to delete a character/digit.

In editing/pre-dial mode: Press and hold to delete all the characters/digits.

During a call: Press to mute/resume the microphone.

During ringing: Press to silence the handset ringer.

In idle mode: Press to intercom another handset.

#### 12. TALK OFF ( )

During a call: Press to end a call and go back to idle screen In menu/editing mode: Press to go back to previous menu.

In menu/editing mode: Press and hold to go back to idle screen.

In idle mode: Press and hold to power off the handset.

Press and hold to power on when it is off.

#### 13. ALPHANUMERIC KEYPAD, STAR (\*), HASH (#)

In predial or editing mode: Press to insert a digit / character /\*/#.

\* key in idle mode: Press and hold to enable or disable the keypad lock.

# key in idle mode: Press and hold to turn on/off the ringer.

0 key in idle/pre-dial mode/number editing mode: Press and hold to insert a pause.

#### 14. R

During a call: Press to answer a new call if you subscribed to Call Waiting service from your service provider.

#### 15. PHONEBOOK ( )

In idle mode: Press to access the phonebook.

**The phone settings are always set in the same manner:**

Press **MENU / OK** ( ) button to open the main menu. Then use the **▼▲** buttons to select an entry and confirm it by pressing the **MENU / OK** ( ) button.

The factory preset PIN code is **0000**

### Example:

#### Set the language on the base

Press **MENU / OK** ( ), then select **▼▲** language, press **MENU / OK** ( ), then select **▼▲** desired language and then press **MENU / OK** ( ) to confirm.

#### Set the language on the handset

Press **MENU / OK** ( ), then select **▼▲** HS settings, press **MENU / OK** ( ), then select **▼▲** language, press **MENU / OK** ( ), then select **▼▲** desired language and then press **MENU / OK** ( ) to confirm.

## MENU SETTING

### Handset

#### PHONEBOOK

##### ►VIEW LIST

under an entry: ►ADD ►VIEW ►EDIT ►DELETE ►DELETE ALL ►PB STATUS ►FASTDIAL

#### CALL LIST

##### ►VIEW LIST

under an entry: ►ADD TO PB ►DELETE ►DELETE ALL ►DETAILS ►CATEGORY

#### BS SETTINGS

►BS RINGER ►RING VOLUME ►DELETE HS ►DIAL MODE ►FLASH TIME ►CHANGE PIN

#### HS SETTINGS

►RING SETUP ►TONE SETUP ►SELECT BASE ►LANGUAGE ►RENAME HS ►HS DISPLAY ►AUTO ANSWER

#### DIRECT MEM

►M1-M2-M3-M4

#### DATE & TIME

►DATE FORMAT ►TIME FORMAT ►SET TIME ►SET DATE

#### ALARM

►ON ►OFF

#### REGISTRATION

►BASE 1, 2, 3 or 4

#### DEFAULT

►PIN? 0000 by default

#### ANS. MACHINE

►MSG PLAYBACK ►DELETE ALL ►MEMO ►TAM ON/OFF  
 ►TAM SETTINGS (ANSWER MODE, TAM LANGUAGE, OGM  
 SETTINGS,ANSWER DELAY, RECORD TIME, COMPRESSION,  
 BS SCREENING,REMOTE ACC, CHANGE PIN)

## HANDSET REGISTRATION

**Note:** Handset and base station are pre-registered.

Up to five handsets can be registered to a single base station.


In the case of re-registration, the PIN code is **0000**.

### Registration a handset

**On Handset:** Press **MENU/OK**  select ▼▲ REGISTRATION

Press **MENU/OK**  select ▼▲BASE(1-4), Press

**MENU/OK**  enter 4 digits PIN for the base PIN.

At the same time on **Base:** press and hold  /**HANDSET  
 LOCATOR** on the base to enter the registration mode.

After a short delay, if the registration is successful, there will be a confirmation beep, and the handset display will go back to idle mode with the new handset number shown. The base display will also go back to idle mode.


### De-Registration

**On Handset:** Press **MENU/OK**  select ▼▲BS SETTINGS,

press **MENU/OK**  select ▼▲DELETE HS, press **MENU/OK**


 enter PIN **0000**, press **MENU/OK**  select ▼▲one of

the handsets that you want to de-register, press **MENU/ OK**

 to confirm.

## INTERCOM

### Internal Call

Press **C / MUTE**  Enter the desired handset number on keypad.



#### NOTE:

If only one cordless handset is registered to the base station, internal call will not be available.

### Call All Handsets

Press **C / MUTE**  Enter "9" on keypad.



### Transfer an External Call

Press **MENU / OK**  select ▼▲INTERCOM, Press **MENU / OK** 


Enter the desired handset number on keypad.

After connection quit the call on handset.

### Conference Call

Press **MENU / OK**  select ▼▲INTERCOM, Press **MENU / OK** 

Enter the desired handset number on keypad.

After connection press and hold  until CONFERENCE is shown.

#### NOTE:

Any party hangs up during a conference call will leave the other party still in connection with the external call.


## PHONEBOOK

Your phone can store up to 50 private phonebook in the cordless handset and the telephone base with names and numbers. Each phonebook entry can have a maximum of 20 digits for the phone number and 12 characters for the name. You can also select different ringer tones for your phonebook entries.


Phonebook entries are stored alphabetically by name.

### To Add new entry


In idle mode :


Press **MENU / OK** , select ▼▲PHONEBOOK to access the phonebook list ( or empty),

Press **MENU / OK**  to select ADD.

Press **MENU / OK**  to select NAME ,enter the name.


Press **MENU / OK**  to select NUMBER, enter number.


Press **MENU / OK**  and ▼▲ to select the desired ringtone for your phonebook entry.

Press **MENU / OK**  to save.

## DIRECT MEM

Your AMPLIDECT295 base can also paired as additional handsets with some specific handset/device like our PhotoDECT295 or EMERGENCY295 which need direct memory numbers programed in the base, and these numbers ( 4 maxi ) have to be added through the regular handset.


**On Handset :** Press **MENU / OK** , select ▼▲DIRECT MEM,

Press **MENU / OK**  select ▼▲M1-M2-M3-M4,

Press **MENU / OK**  enter number then OK.


## ANSWERING MACHINE

Your telephone base includes a telephone answering machine that records unanswered calls when it is on. The answering machine can store up to 59 messages within the maximum recording time of approximately 15 minutes. As well as recording incoming messages, you can record memos for other users of your phone.

Press  /**ANS ON/OFF** to switch the answering machine ON or OFF.

### REMOTE OPERATIONS

If you are away from your home, you can call your answering machine to switch on or off the answering machine and listen to your messages remotely using another tone-dialing phone.

1. Place a call from a tone-dialing phone.
2. When the answering machine answers the call and starts playing OGM, press .
3. Enter the 4-digit Remote Access PIN (default PIN is **0000**).
4. Press the following keys to carry out your desired function.

key	Functions	
	While message is not playing	While message is playing
2		Delete the current message playback
4		Repeat playing the current message from the beginning. Press twice to skip backward to the previous message.
5	Play the message	
6		Skip forward to play the next message
7	Turn on the answering machine	
8		Stop the current message playback
9	Turn off the answering machine	

*	Enter the 4-digit remote access PIN	
---	-------------------------------------	--

#### NOTE:

When your answering machine memory is full and the answer mode is in ANS & RECORD, the answer mode will change to ANSWER ONLY automatically.

It will change to ANS & RECORD mode automatically after some messages have been deleted.

You cannot delete new messages before they were listened.

## TECHNICAL SPECIFICATIONS

Basic Information	
Valid Range	Indoor : 50m Outdoor :300m
Handset Operating time	Standby time : 100 hours Talk time : 10 hours approx.
RF Parameter	
Frequency	1.88 GHz to 1.9GHz
Nb Channels	10
Power Supply	
Base	Input 100-240V 50/60Hz; output 6V; 450mA adapter
Charger	Input 100-240V 50/60Hz; output 6V; 450mA adapter
Handset	2.4V ; 500mAH rechargeable NIMh battery pack

## MAINTENANCE

Clean the surface of the housing with a soft, fluff-free cloth. Do not use any cleaning agents or solvents.

## GUARANTEE

From the moment your Geemarc product is purchased, Geemarc guarantee it for the period of two years. During this time, all repairs or replacements (at our discretion) are free of charge. Should you experience a problem then contact our help line or visit our website at [www.geemarc.com](http://www.geemarc.com).

The guarantee does not cover accidents, negligence or breakages to any parts. The product must not be tampered with or taken apart by anyone who is not an authorized Geemarc representative.

The Geemarc guarantee in no way limits your legal rights.

**IMPORTANT:** YOUR RECEIPT IS PART OF YOUR GUARANTEE AND MUST BE RETAINED AND PRODUCED IN THE EVENT OF A WARRANTY CLAIM.

**Please note: The guarantee applies to the United Kingdom only.**

## DECLARATIONS:

 **CE Directives**

Geemarc Telecom SA hereby declares that this product is in compliance with the essential requirements and other relevant provisions of the Radio and Telecommunications Terminal Equipment Directive 2014/53/UE and in particular article 3 section 1a,1b and section 2. The telephone does not operate if the line current is lower than 18mA.

The CE declaration of conformity may be consulted at [www.geemarc.com](http://www.geemarc.com)

**Electrical connection:** The apparatus is designed to operate from a 230V 50Hz supply only. (Classified as hazardous voltage

according to EN60950 standard). The apparatus does not incorporate an integral power on/off switch. To switch off the apparatus, disconnect the power adaptor.

When installing the apparatus, ensure that the mains power socket is readily accessible.

### Telephone connection:

Voltages present on the telecommunication network are classified TNV-3 (Telecommunication Network Voltage) according to the EN60950 standard.



### Recycling Directives

The WEEE (Waste Electrical and Electronic Equipment) has been put in place for the products at the end of their useful life are recycled in the best way.

When this product is finished with, please do not put it in your domestic waste bin. Please use one of the following disposal options:

- Remove the batteries and deposit them in an appropriate WEEE skip.

Deposit the product in an appropriate WEEE skip.

- Or, hand the old product to the retailer. If you purchase a new one, they should accept it.

Thus if you respect these instructions you ensure human health and environmental protection.



For product support and help visit our website at

[www.geemarc.com](http://www.geemarc.com)  
 telephone **01707 387602**  
 or fax **01707 832529**