**Amplified hearing assistant  
Amplificateur portatif  
Tragbarer Höverstärker  
Amplificator portatil**

**English P1**

**Français P16**

**Deutsch S30**

**Español P44**

Please refer to our website:  
[www.geemarc.com](http://www.geemarc.com) for an up to date user  
guide, as there may be important updates and  
changes you need to be aware of.

<b>CONTENTS</b> .....	<b>2</b>
<b>INTRODUCTION</b> .....	<b>3</b>
Unpacking the LoopHEAR™10 .....	4
<b>DESCRIPTION</b> .....	<b>5</b>
General Description .....	5
<b>INSTALLATION</b> .....	<b>8</b>
Setting Up .....	8
<b>USING THE LoopHEAR™10</b> .....	<b>9</b>
Turning the LoopHEAR™10 On and Off .....	9
Tone and Balance Setting .....	10
<b>TROUBLESHOOTING</b> .....	<b>11</b>
<b>SAFETY INFORMATION</b> .....	<b>12</b>
<b>GUARANTEE</b> .....	<b>13</b>
<b>FCC STATEMENT</b> .....	<b>14</b>
<b>RECYCLING DIRECTIVES</b> .....	<b>15</b>

# INTRODUCTION

Congratulations on purchasing your Geemarc LoopHEAR™10. The LoopHEAR™10 is a rechargeable assistive listening device. It amplifies speech together with sounds from a variety of sources like TV, radio and telephone.

The LoopHEAR™10 can be used as an alternative to Hearing Aids when using headphones.

**Note:** users of digital hearing aids must make sure their aids have been programmed for use with the 'T' setting.

It is important that you read the instructions below in order to use your Geemarc LoopHEAR™10 to their full potential. Keep this user guide in a safe place for future reference.

## Unpacking the LoopHEAR™10

---

When unpacking the LoopHEAR™10, you should find the following in the box:

- LoopHEAR™10 Conversation Piece
- Headset with rubber ear cushions
- 3.7V Li-po Rechargeable Battery
- Micro usb / usb cable for recharging
- Lanyard (**option**)
- A microphone with "holder" and 3m lead (**option**)
- A belt clip
- 1 User Guide

For product support and help visit our website at

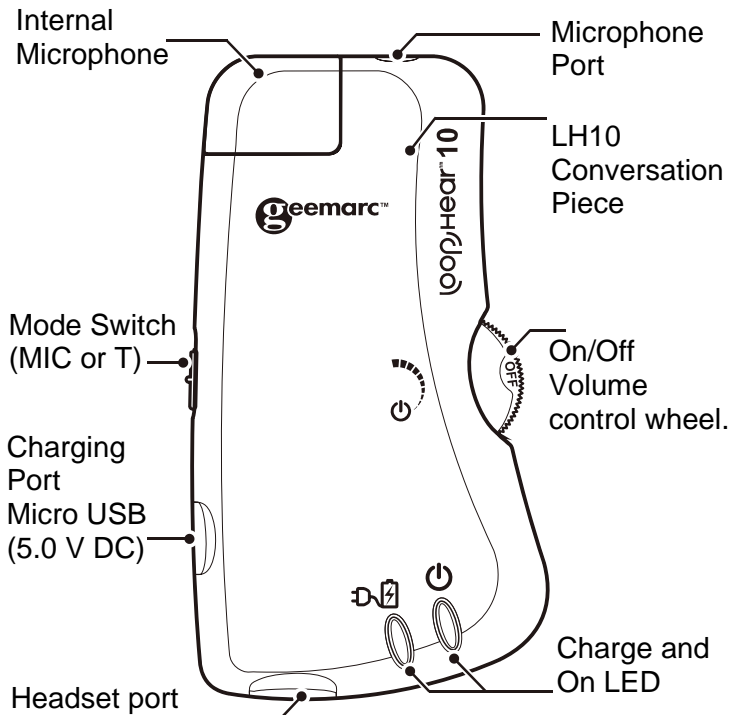
**[www.geemarc.com](http://www.geemarc.com)**

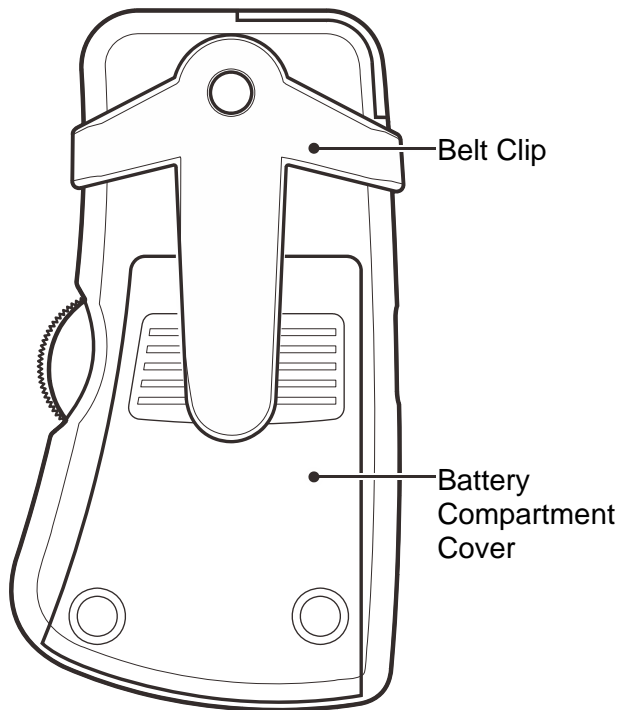
telephone **01707 387602**

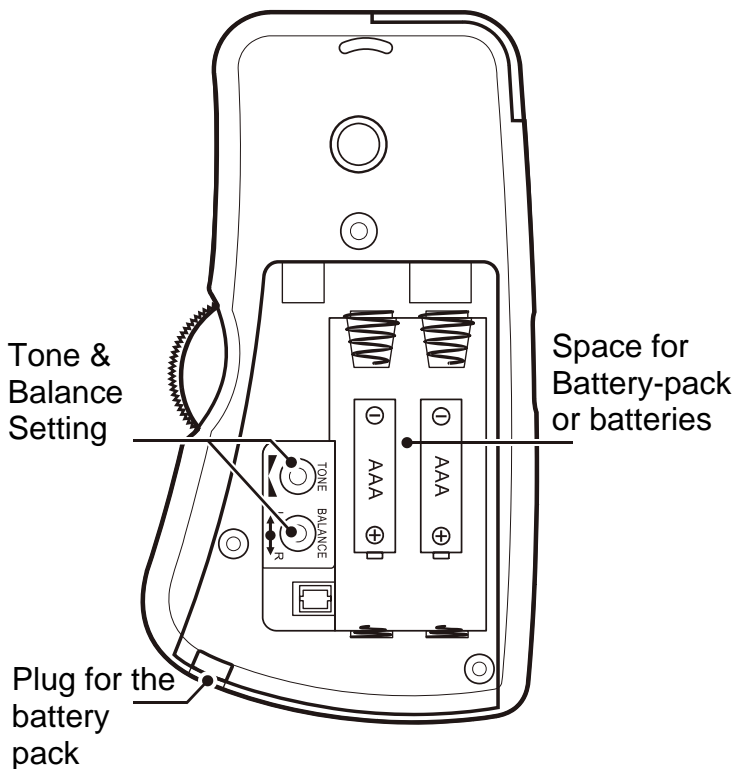
or fax **01707 832529**

## General Description

---

## Setting Up

---

Open the battery compartment. Connect the supplied rechargeable battery (3.7V) into the battery compartment located on the rear of the LH10 Conversation Piece.

Or insert 2 single batteries (1.5V AAA non rechargeable) instead of the set battery supplied.

With AAA battery mode, if the 'On' LED turns red when in use, the LH10 needs to have new batteries ( No Red LED indication when using the rechargeable battery).

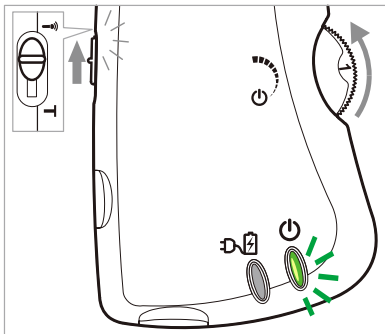
To charge the battery, connect the supplied USB cable to the micro USB socket on the left hand side of the unit and then into the USB to PC or standard power supply 5V DC. The power supply LED turns red..

## Headset

Insert the headset lead into the headset port found on the base of the Communication Piece.

## Turning the LoopHEAR™10 On and Off


Switch the headset on. Ensure the mode switch is set to  (MIC). Roll the On/Off Volume control wheel upwards. The On LED will display Green. Adjust the Volume to a comfortable level by rolling the On/Off Volume control wheel.

To turn the LH10 off or on, roll the On/Off Volume control wheel downwards. The Green LED will switch off and OFF will be showing on the On/Off Volume control wheel

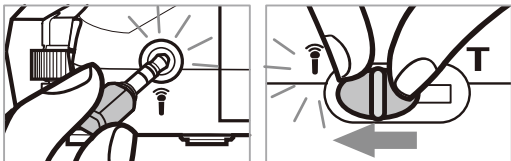The LH10 can be used in rooms especially equipped with a magnetic loop (conference ...etc) due to its "built in T coil". Simply by setting the mode switch to **T**, the LH10 will receive the emitted magnetic waves then amplify them to your headset. You will receive all the benefits of the "built in T coil".

An external microphone (option) can be connected in the socket  "Mic", then set the switch to  "Mic" position, in this case the "built in T coil" is disabled.


## Microphone

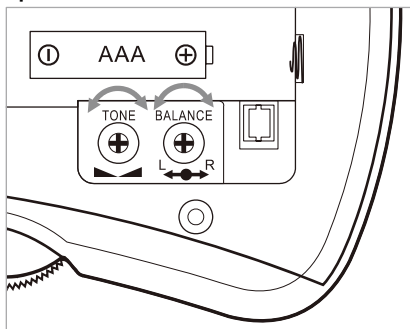The LH10 is built with an internal microphone which enables you to hear all the surrounding sounds.

## Tone and Balance Setting

The tone () and Balance () can be adjusted by using the controls found within the battery compartment.

## **The 'On' LED is Red**

- Charge the battery if the battery pack is used or replace with new batteries if standard AAA batteries are installed.

## **The LH10 appears not to be working**

- Make sure that the headset is securely plugged in
- Make sure that the volume is correctly adjusted
- Make sure that the battery is fully charged or use good batteries.
- Make sure the mode switch is set to MIC if using headset/earpiece or T if using an induction loop.

## **The LH10 Is not charging**

- Make sure that the power supply cord is plugged in correctly i.e. into a working wall socket and into the LH10 Conversation Piece
- Make sure that the battery is inserted correctly and is of the recommended type

## General

Do not open the unit. Contact the helpline for all repairs.

## Cleaning

Clean the LH10 with a soft cloth. Never use polishes or cleaning agents - they could damage the finish or the electrics inside.

## Environmental

Do not expose to direct sunlight.

Always ensure there is a free flow of air over the surfaces of the LH10.

Do not place any part of your product in water and do not use it in damp or humid conditions e.g. bathrooms.

Do not expose your product to fire or other hazardous conditions.

Avoid touching the LH10 Conversation Piece with clothing or hands, as this can create excessive noise.

# GUARANTEE

From the moment your Geemarc product is purchased, Geemarc guarantee it for the period of 2 years. During this time, all repairs or replacements (at our discretion) are free of charge. Should you experience a problem then contact our help line or visit our website at **[www.geemarc.com](http://www.geemarc.com)**. The guarantee does not cover accidents, negligence or breakages to any parts. The product must not be tampered with or taken apart by anyone who is not an authorised Geemarc representative. The Geemarc guarantee in no way limits your legal rights.

**IMPORTANT:** YOUR RECEIPT IS PART OF YOUR GUARANTEE AND MUST BE RETAINED AND PRODUCED IN THE EVENT OF A WARRANTY CLAIM.

**DECLARATION:** Geemarc Telecom SA hereby declares that this product is in compliance with the essential requirements and other relevant provisions of the Radio and Telecommunications Terminal Equipment Directive 2014/30/UE.

*The declaration of conformity may be consulted at [www.geemarc.com](http://www.geemarc.com)*

## FCC CLASS B

**NOTE:** This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of FCC Rules.

These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications.

However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Re-orient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio TV technician for help.

**NOTICE:** This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) this device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

The WEEE (Waste Electrical and Electronic Equipment) has been put in place for the products at the end of their useful life are recycled in the best way.

When this product is finished with, please do not put it in your domestic waste bin.

Please use one of the following disposal options:

- Remove the batteries and deposit them in an appropriate WEEE skip. Deposit the product in an appropriate WEEE skip.
- Or, hand the old product to the retailer. If you purchase a new one, they should accept it.

Thus if you respect these instructions you ensure human health and environmental protection.

All electrical and electronic products including batteries should be disposed of separately from the municipal waste stream via designated collection facilities appointed by the government or the local authorities.

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